天滌・BIO SANITIZER

自動感應手部消毒機
Auto Hand Disinfector

AD2300

使用說明書
User Manual

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Thank you for choosing AQ

The AD2300 Auto Hand Disinfecter is going to provide users unprecedented personal hygiene experience.





The machine is designed to maximize the effectiveness of AQ Bio Sanitizer.

PLEASE READ THIS MANUAL THOROUGHLY BEFORE USE

Index


Parts Identification	○
Installing the batteries	○
Liquid Refill	○
Operation Instruction	○
Performing hand hygiene	○
Specifications	○

Parts Identification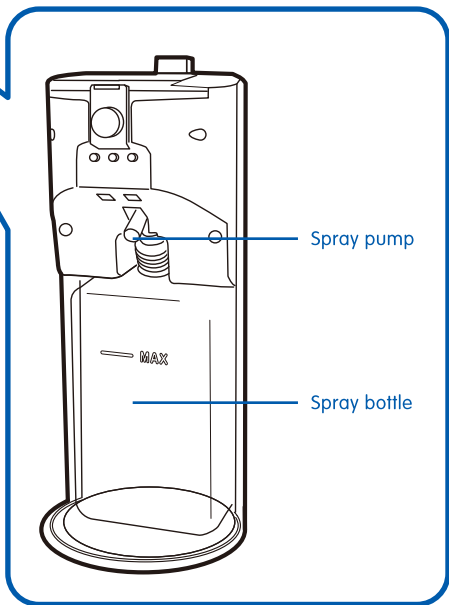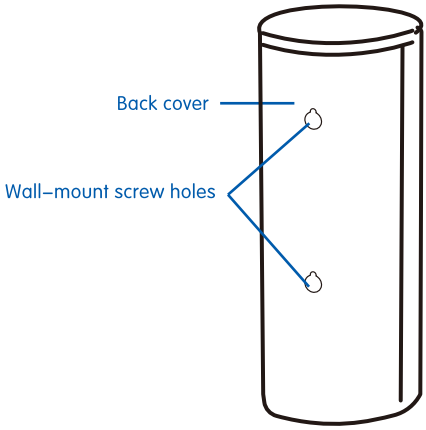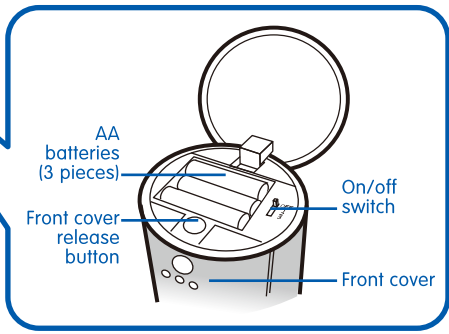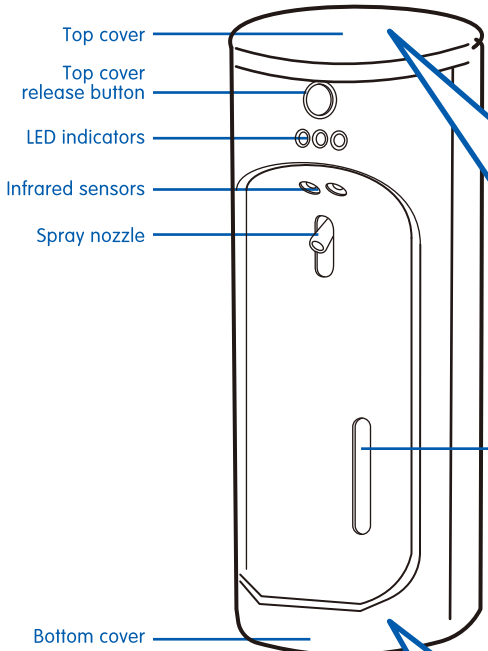

Installing the batteries


- 1. Press the top cover release button to flip open the top cover
- 2. Install 3 AA size ALKALINE batteries
- 3. Change to new batteries if the red LED light is lit
- 4. Remove the batteries if the machine is not used for a prolong period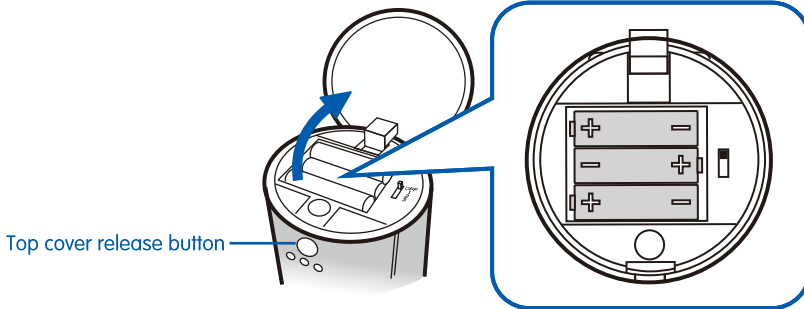

Liquid Refill


1. Remove the spray bottle

- 1. Open the front cover of the machine by pressing the front cover release button
- 2. Remove the spray bottle gently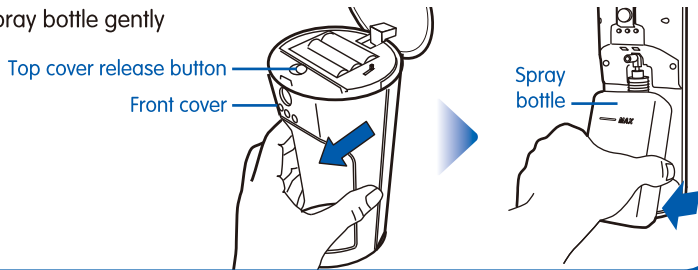

2. Fill in AQ Bio Sanitizer

- 1. Fill in AQ Bio Sanitizer to no more than the MAX line
- 2. Reinsert the spray bottle
- 3. Put the front cover back on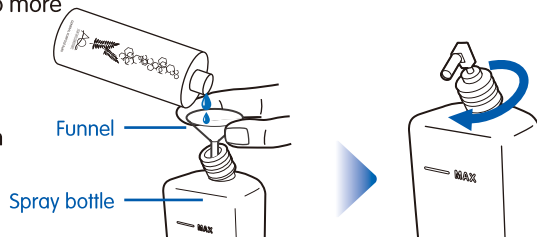


Operation Instruction

- 1. Open the top cover
- 2. Select 1, 2 or 3 times consecutive spray action by switching the ON/OFF button to the desired position
- 3. The machine will now calibrate itself and the LED lights will blink for a few seconds
- 4. When the slow blink green LED comes on, the machine is ready for operation
- 5. Close the top cover


Consecutive spray option

- OFF** Switch off the Hand Disinfectant
- 1** Spray once at a time for smaller or children's hands
- 2** Spray twice in a role for adult's hands, one time for the left hand and one time for the right
- 3** Spray thrice in a role for serious hand hygiene or to remove stubborn odour, one time for the left, the right and one time for both hands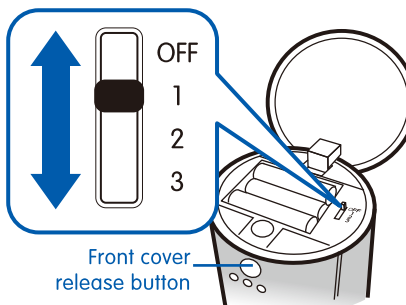


LED indicators

- Red** Change to new batteries if the red LED light is lit
- Green** When the slow blink green LED comes on, the machine is ready for operation
- Blue** The blue LED will blink just before the spray head releases AQ Bio Sanitizer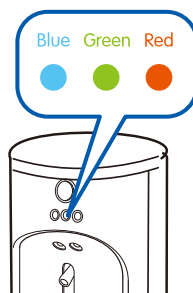

Performing hand hygiene

- Place your hands within 20 cm in front of the infrared sensors to receive the preset portion of sanitizer, approx. 0.1 ml per spray
- The blue LED will blink just before the spray head releases AQ Bio Sanitizer

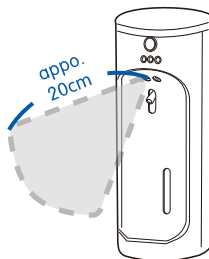Perform hand hygiene thoroughly as shown by the following diagram

- Band hands slightly to receive the sanitizer

- Rub palms against each other

- Rub fingertips against the centre of the palm

- Clean back of hand with palm

- Cross the fingers to clean between finger joints

- Hold and twist clean the thumbs

- Clean wrist carefully

- Rub hands until the sanitizer dries up

Repeat if necessary

Specifications

Model No	AD2300
Dimension	9cm (L) x 23cm (H) x 9cm (W)
Voltage	DC4.5v 3 AA size ALKALINE batteries
Bottle Capacity	approx. 300ml ("MAX" mark on the bottle)
Net Weigh	475g (not including batteries and spray bottle)

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發票號碼 機身編號(如適用)
Invoice No. Serial No. (If applicable)

(當產品需要維修保養服務時，必須提供發票正本)
(The original invoice must be presented when maintenance service is required)

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- Your registered product is entitled to a 12-month warranty period from the original date of purchase.
- The original invoice (with date of purchase clearly indicated) for the product must be provided when making any claim under this warranty.
- The user must follow the guidance of the user menu to operate the product. All damages caused by misuse will not be covered by this warranty.
- This warranty does not cover:
 - Product not sold by AQ or an authorized distributor.
 - Damages caused by water, fire, accident, negligence, abuse or misuse, improper liquid or voltage input, rechargeable battery problems, normal wear and tear, any unauthorized repair or being tempered with.
- Should the product require repair services, the customer is responsible for the transportation cost to and from the service centre.
- AQ Bio Technology Group Ltd reserves the right to amend or cancel any of the terms and conditions of this warranty without prior notice.
- In case of dispute, AQ reserves the right to final decision.

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Please fill out this form and submit it by post marked with "AQ Product Warranty Registration" to:

AQ Bio Technology Group Ltd, 8/F., Contempo Place, 81 Hung To Road, Kwun Tong, Kowloon, Hong Kong.

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AQ Bio Technology Manufacturing Ltd.